SRX820 · 1

SRX820 · 2


SRX820 · 3

© Copyright MediRoyal Nordic AB
Sweden · 2024

P/N 5105132 · REV 2024/03
26202DR · www.vajjer.se

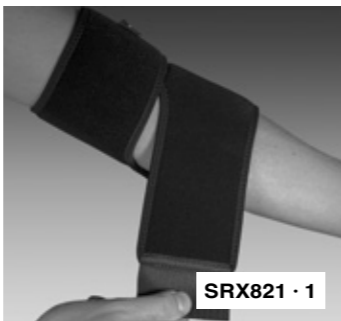SRX821 · 1

SRX821 · 2

SRX822 · 1

SRX822 · 2

SRX® by Mediroyal

– Support during Activity, taking you further.

The SRX® line has been designed to support people in activity. The new innovative SRX® material has a 3-dimensional top surface textile that provides both elasticity and compression. The core is made from CR-foam that contains closed micro air-cells to provide a lightweight material with good heat insulation and no moisture absorption. Our Neptune™ material, which is hydro-dynamic, comfortable and quick-drying, has been laminated on the inside.

The new materials, in combination with a functional design, and our Scandinavian heritage that values an active lifestyle, are all essential parts of our SRX® line. Support during Activity, taking you further.

MediRoyal Nordic AB · Box 7052 · SE-192 07 Sollentuna · Sweden
+46 8 506 766 00 · www.mediroyal.se

mediroyal

SRX®

Elbow

GB English – Instructions

General information · This Mediroyal SRX® product has been designed to support you during your active lifestyle. If you got an injury, consult a doctor or therapist for an individual rehabilitation plan. If you suffer from poor circulation, reduced sensibility, or diabetes, be observant on pressure marks or local swelling when you use the product. If your condition gets worse when using the product, discontinue the use and contact a doctor. Follow the washing instruction on the product label. If you have material allergies you can find the material composition on the label. For more information, visit www.mediroyal.se

Instruction · The product has been designed to support the elbow and provide local heat. Pull the brace up over the arm and position it over the elbow joint. Close the hook and loop.

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