
mediroyal

Orthoses is our passion


Instruction Overlap Thumb Christine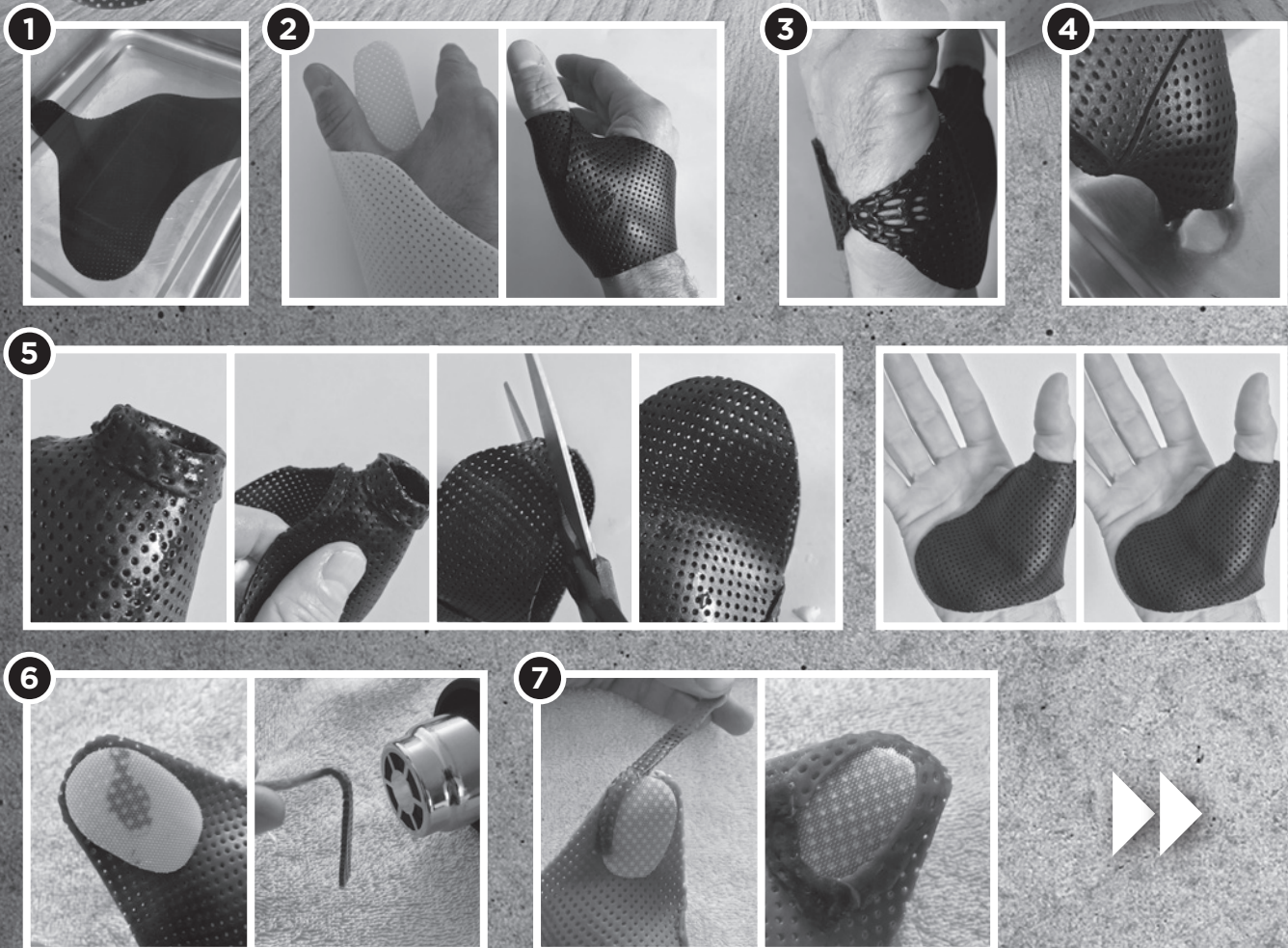

Mediroyal Nordic AB · Box 7052 · 192 07 Sollentuna · Sweden · Tel +46 8 506 766 00
info@mediroyal.se · facebook.com/Mediroyalnordic · www.mediroyal.se

8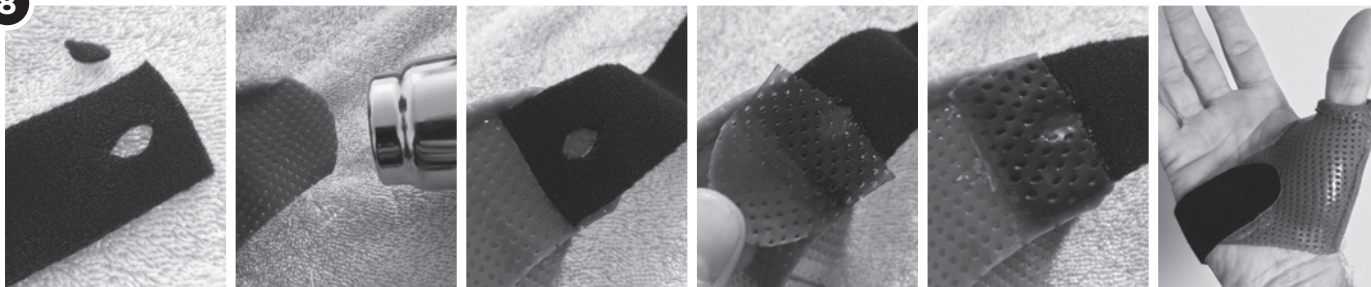

GB

English – instructions

Instruction Overlap Thumb Christine

Before application, if you use the lined Comfort quality, remove the lining of the inside of the thumb tab that will be used to overlap. This can be done by scratching it with blade knife. Otherwise it might be difficult to bond it to the surface.

- 1 Immerse the pre-cut into the water and let it stay for about one minute. Carefully lift it up and place it quickly on a dry towel. Move it from the towel onto the patient.
- 2 Position the pre-cut so the thumb flap overlaps from the inside of the palm to the dorsal side of the thumb. Carefully attach the material together by stretching the thumb flap and attaching it to the surface of the plastic. Press the surface slightly to get a permanent bond.
- 3 While the thermoplastic is still soft, position the thumb into an opposition grip so the tip of the thumb can reach the tip of the Dig II. Then attach the two arms of the brace on the ulnar side by pressing the material together.
- 4 When the thermoplastic has cooled, open the ulnar side by separating the material and slide the thumb splint off the thumb.
- 5 Adjust the length of the thumb piece by immersing the tip into the water and then fold it circular. Then adjust the length of the two arms by cutting them with a scissor. Round the ends for a better comfort.

To shape the pre-cut individually you have to use hot water. Recommended temperature is 65–75° C.

Adjust the web-space area by immersing it into water and then roll it outwards.

- 6 Attach the hook closure on the part inside of the hand. Cut a hook piece. If not self adhesive, spot heat the surface of the arm for a few seconds then position the hook part and press slightly.
- 7 The hook part can be welded in the edge by using a strip of thermoplastic. Cut a 3–4 mm wide strip that is 6–8 cm long. Heat it with a heat gun and then attach the adhesive strip around the hook and then work the thermoplastic into the hook part and the main body.
- 8 The loop strap can be permanently welded onto the dorsal flap. Cut a loop strap in the desired length. Use a heat gun to melt the loop on the first 2–3 cm of the strap. Once it has cooled, cut a small hole in the center of the strap. Spot heat the area where the strap should be positioned onto the flat and press it into place. Then let the flap cool. In order to make the attachment stable, cut a thermoplastic piece that is 2–3 mm wider than the strap and 4–5 cm long. Spot heat the first 2 cm of the strip and when soft you position that over the attached loop piece. Cut away the excess part of the strip and work the hot thermoplastic over the strap, pressing the area over the hole in the strap. That creates a thermoplastic weld.

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