© Copyright MediRoyal Nordic AB · Sweden · 2021

P/N 5105143 · REV 2021/06 · 25886DR · www.vajer.se

GB English – Instructions

General information · This Mediroyal SRX® product has been designed to support you during your active lifestyle. If you got an injury, consult a doctor or therapist for an individual rehabilitation plan. If you suffer from poor circulation, reduced sensibility, or diabetes, be observant on pressure marks or local swelling when you use the product. If your condition gets worse when using the product, discontinue the use and contact a doctor. Follow the washing instruction on the product label. If you have material allergies you can find the material composition on the label. For more information, visit www.mediroyal.se

Instruction · The Ice Wrap includes an ice container that should be filled with ice and water and a wrap that holds it in place. It has been designed to be used after trauma in order to administrate cold therapy. Never use the ice bag directly on the bare skin as it might cause a frost bite. We recommend using a towel or cloth in between. Check the skin frequently for discoloring or blisters. Remove the ice bag if you notice skin changes, loss of sensation or feel a burning numbness in the area. Limit the use of the Ice Wrap for 20-30 minutes per treatment session. The wrap can be used for knee, elbow or shoulder.

Precautions · If you have diabetes, circulatory problems or sensitive skin you should consult your doctor before use. If you are allergic, check the material content on the textile label before use. For more information visit www.mediroyal.se

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SRX® by Mediroyal

– Support during Activity, taking you further.

The SRX® line has been designed to support people in activity. The new innovative SRX® material has a 3-dimensional top surface textile that provides both elasticity and compression. The core is made from CR-foam that contains closed micro air-cells to provide a lightweight material with good heat insulation and no moisture absorption. Our Neptune™ material, which is hydro-dynamic, comfortable and quick-drying, has been laminated on the inside.

The new materials, in combination with a functional design, and our Scandinavian heritage that values an active lifestyle, are all essential parts of our SRX® line. Support during Activity, taking you further.

MediRoyal Nordic AB · Box 7052 · SE-192 07 Sollentuna · Sweden
+46 8 506 766 00 · www.mediroyal.se

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