**General information**  
Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the textile label.

**Indications**  
The product has been designed to provide support and immobilization of the shoulder and elbow after soft tissue injuries.

**Contraindications**  
Patients with instable fractures should not use this product unless the fracture has been stabilized.

**Pre Application**  
Select the right size using the sizing reference on the measurement chart.

**Adjustment**  
The length of the pocket is important for the support of the sling. Make sure to select the right size, rather a little too long than too short. The strap around neck is adjustable by the hook strap. If your product is fitted with a belt, it's also adjustable by the hook straps.

**Application**  
Open the hook strap and slide the arm into the pocket. Adjust and secure the hook strap around the neck. If the product is fitted with a waist band adjust it and secure the hook. The product has been fitted with a thumb loop to provide additional support.

**Removal**  
Open the straps and remove the sling.

**Caring instructions and precautions**  
We recommend that the brace should be washed once a week. Follow the washing instructions on the white label on the product, close all straps and use a laundry bag.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the label.

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© Copyright MediRoyal Nordic AB, Sweden - 2016 - 25076DR - [www.vajer.se](http://www.vajer.se)