© Copyright MediRoyal Nordic AB, Sweden · 2022 · 25991DR · www.vajer.se

GB English – Instructions

Omox Shoulder Immo

General information · Read the instruction carefully. If you got an injury, you should consult with a doctor or therapist for an individual rehabilitation plan. If you suffer from poor circulation, reduced sensibility, or diabetes, be observant on pressure marks or local swelling when you use the product. If your condition gets worse when using the product, discontinue the use and contact a doctor. If the product has been fitted by a medical trained professional like occupational therapist, physical therapist, orthopedic technician or a medical doctor, please refer to them for advice and further instructions. Follow the washing instruction on the product label. If you have material allergies you can find the material composition on the label. For more information, visit www.mediroyal.se

Indications · The product has been designed to provide support and immobilization to the shoulder post-surgery.

Contraindications · Patients with instable fractures should not use this brace unless the fracture has been stabilized first.

Pre Application · The product is made universally sized and works for both left and right. There is an arm pouch and a detachable body strap.

Application · Open the straps on the arm pouch. Adjust the length of shoulder straps and secure them in the buckles so they can not run. Position the arm into the pouch and run the straps over the shoulders. Criss-cross the straps in the back and run then on each side of the body and attach them onto the pouch. Adjust the length of the straps with the buckles if necessary. Secure the pouch with the body strap. This will prevent external rotation of the arm and shoulder.

Removal · Remove the body belt first, then open the straps on the pouch.

Material content · Check the material content on the white label.

Caring instructions and precautions · We recommend that the brace should be washed once a week. Follow the washing instructions on the white label on the product, close all straps and use a laundry bag. Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on label.

For more information visit www.mediroyal.se

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mediroyal

mediroyal

Mediroyal – Functional Swedish Design

Product development is a combination of ideas, material, design and know-how. At Mediroyal we have more than 20 years of experience with developing products that provides both the patient and the therapist with enhanced functionality, every day. Swedish design is unique in many ways. Quality, performance, functionality and design are all features that make a difference. That's why we are proud to label our products with "Functional Swedish Design". Because orthoses are our passion.

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