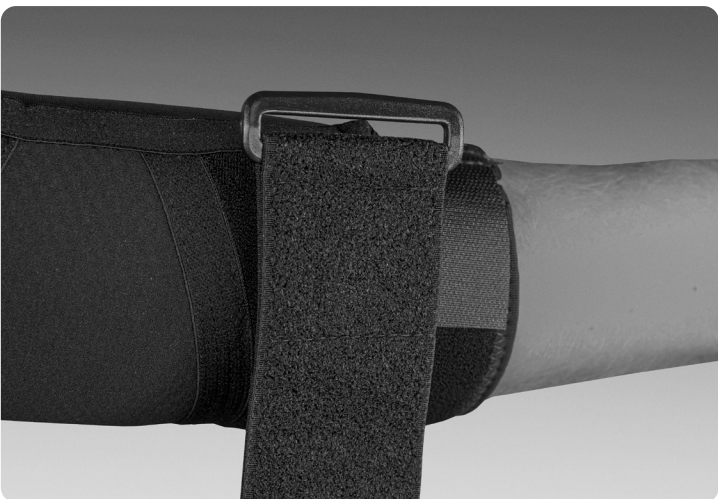
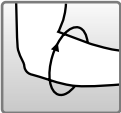
Size chart

Size	XS	S	M	L	XL	XXL
cm	20–23	23–25	25–28	28–31	31–33	33–36

© Copyright MediRoyal Nordic AB, Sweden · 2021 · 25875DR · www.vajer.se

English – instructions

NRX Epi Elbow Brace

General information · Read the instruction carefully. If you got an injury, consult a doctor or therapist for an individual rehabilitation plan. If you suffer from poor circulation, reduced sensibility, or diabetes, be observant on pressure marks or local swelling when you use the product. If your condition gets worse when using the product, discontinue the use and contact a doctor. Follow the washing instruction on the product label. If you have material allergies you can find the material composition on the label. For more information, visit www.mediroyal.se

Indications · Lateral or medial epicondylitis.

Pre Application · To select the right size, take circumference measurement 5 cm below the elbow joint. Use the measurement chart to determine the right size.

Adjustment · The pad on the brace should be placed over the highest position of the muscle. If you have lateral epicondylitis the pad should be placed on the upper part of the arm. If you have medial epicondylitis, place the pad on the lower side of the arm.

Application · Open the Velcro strap. Position the pad over the highest part of the muscle. Close the strap but do not over tighten. When you make a fist with your hand the pressure from the pad should be felt over the muscle. The brace is only effective during loading of the hand/arm. It shouldn't be used when you rest or sleep. Epicondylitis are usually caused by over extension or wrong position of the wrist. Therefore it might be a good idea to combine the brace with a wrist brace as well. This unloads the tendons and decreases the pain.

Removal · Open the Velcro strap and remove the brace. Always then check for pressure marks.

Caring instructions and precautions · We recommend that the brace should be washed once a week. Follow the washing instructions on the label and always use the laundry bag. Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the label.

More information can be found on www.mediroyal.se

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Mediroyal – Functional Swedish Design

Product development is a combination of ideas, material, design and know-how. At Mediroyal we have more than 20 years of experience with developing products that provides both the patient and the therapist with enhanced functionality, every day. Swedish design is unique in many ways. Quality, performance, functionality and design are all features that make a difference. That's why we are proud to label our products with "Functional Swedish Design". Because orthoses are our passion.

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