

DEX® and NRX® Strap



The DEX® line has a loop surface that makes it possible to attach NRX® Strap. It makes it possible to use the DEX® wrists as a stable attachment for the NRX® Strap.

Finger Extension

Material

MR2270 DEX® Mid, NRX® Strap 30 mm, NRX® Strap Neptune 50 mm



Use the 60 mm NRX® Strap Neptune for the double extension loop on the MCP. The PIP extension should be made with 30 mm NRX® Strap. More information can be found in the NRX® Finger Extension Assist, Multiple Joints instruction.

Caring instructions

The NRX® strap can be washed in 40 degrees machine wash with a liquid detergent. Always use a laundry bag. Remove the hook parts before washing and let air dry.

Thumb Abduction

Material

MR2274 DEX® Short, NRX® Strap 30 mm



Use 30 mm NRX® Strap to create the thumb abduction support. Alternate the attachment on the DEX® dorsal part for individual support. More information can be found in the NRX® Thumb Abduction Strap instruction.

Finger Flexion

Material

MR2271 DEX® Long, NRX® Strap 110 mm



Use the NRX® Strap in 110 mm width and cut it down to fit the circumference of the finger. Cut a T-shape that wraps around the finger between MCP and PIP. Adjust the length of the strap for an individual flexion pull. More information can be found in the NRX® Finger Flexion instruction.