

NRX® STRAP

- One strap, unlimited options!

NRX® Arch Strap

Material

NRX® Strap 50 mm with 30 mm reinforcement, either NRX® PLUS or NRX® Strap





Cut the length according to the circumference of the foot plus 2 cm. The reinforcement should cover half the length of the main construction.





Apply the strap with the hook running from the outside. Stretch the strap so it supports the arch.





Secure the strap so the hook is positioned over the arch. This way it will provide a slightly lifting to the arch during loading.



Attach the reinforcement strap on the lateral side. Stretch the strap and apply it over the main strap, running into the arch.





Attach the reinforcement strap on the lateral side. Stretch the strap and apply it over the main strap, running into the arch.

Caring instructions

The NRX® strap can be washed in 40° C machine wash with a liquid detergent. Always use a laundry bag. Remove the hook parts before washing and let air dry.

© Mediroyal Nordic AB 2020