

## NRX® STRAP

- One strap, unlimited options!

## **NRX® Foot Drop**

Material

NRX® Strap 110 mm and 50 mm



- A. Cut the ankle part from the 110 mm NRX® Strap, make sure that the straps are long enough to go around the ankle. Usually 40–50 cm is enough.
- B. The cuff should be long enough to wrap around the upper part of the lower leg.
- C. The spiral strap should be at least 50-60 cm long.





Apply the ankle part over the heel with the application running over the lateral part. Apply even tension over the complete strap.







Attach the first strap around the ankle and secure it with the hook. Then apply the second strap, position it closely to the 5th metatarsal and pull evenly over the strap to provide lift to the foot.





Continue with the strap around the ankle and secure it with the hook.

## **Caring instructions**

The NRX® strap can be washed in 40° C machine wash with a liquid detergent. Always use a laundry bag. Remove the hook parts before washing and let air dry.