## GB English – instructions

MediRoyal Posture Support

**General information** · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the backside of the white textile label on the cover for material content.

**Indications** · The product has been design to provide posture support and assistance.

**Contraindications** · Patients with instable fractures should use a more stable brace.

**Pre Application** · To select the right size, take the circumferential measurement around the waist. Use the measurement chart to determine the most suitable size. The brace is adjustable and the size depends sometimes on the tolerance for pressure by the individual.

**Adjustment** · The brace are fitted with sleeves that can be worn over the shoulder straps to reduce pressure in the shoulder/arm pit region. Simply slide them onto the strap before fitting.

**Application** · Open the Velcro closure of the brace. Apply the Y-shaped brace with the two longer straps over the shoulder, see picture. Close the strap around the waist first. Then grip the longer straps, pull them behind your back, exchange each ending of the strap with your hands and pull to the front. It's important to get an even pressure, therefore make sure to stretch the strap fully behind your back. That provides a more even pressure and better support. Close the strap in the front, just under the waist strap.

**Removal** · Open the Velcro straps and remove the brace. It's recommended to close the closures then the brace is not in use. That increases the life expectancy of the brace.

**Caring instructions and precautions** · We recommend that the brace should be washed once a week. Follow the washing instructions on the white label, close the straps and always use a laundry bag.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the backside of the label on the cover.

For more information consult www.mediroyal.se

## FR Français – instructions

MediRoyal Soutien de Posture

The MediRoyal Posture Support brace is made of a soft, elastic fabric with a Y-shaped brace and two longer straps over the shoulder. The brace is adjustable and can be worn over the shoulder straps to reduce pressure in the shoulder/arm pit region.

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