
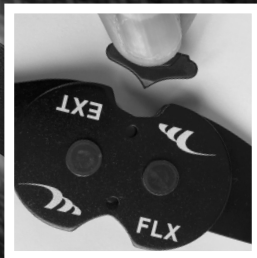
mediroyal 

Knee Hinges by Mediroyal

MR4996 Double Hinge

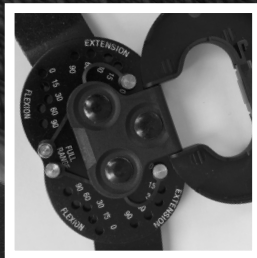MR4997 Active Hinge

MR4998 ROM Hinge

English **GB**

Svenska **SE**

Suomi **FI**

Deutsch **DE**

Français **FR**

Italiano **IT**

Español **ES**

Português **PT**

Nederlands **NL**

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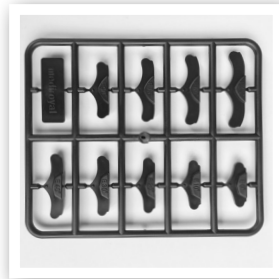
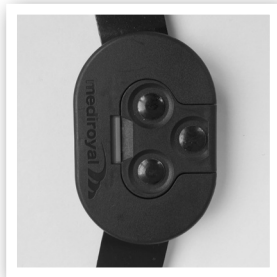
English – instructions

Knee Hinges by Mediroyal

As a part of the update of the knee brace program we have designed three new hinges with different functionality. They are all manufactured of a special aluminum alloy A6061, used in the aircraft industry. The A6061 alloy is strong, lightweight, durable and corrosion resistant.

Technical Data Mediroyal Aluminum A6061:

Density (ρ)	2.70 g/cm ³ [1]
Tensile strength (σt)	124–290 MPa (18.0–42.1 ksi)
Elongation (ε) at break	12-25%
Melting temperature (Tm)	585 °C (1,085 °F)
Thermal conductivity (k)	151-202 W/m*K
Specific heat capacity (c)	897 J/kg*K

Model program

All models have the same dimensions and can be exchanged within the different knee braces. The ROM-hinge is also available in a longer model.

General Precautions · The Mediroyal hinges are all mechanical stabilization elements to further stabilize the knee, primarily in extension and sideways. In order to prevent pressure problems and to get a better stability, always shape the hinge to the contour of the knee.

These hinges are not designed for flexion problem that requires a full body loaded flexion control. In that case, turn to an orthotic workshop for a custom solution.

Mediroyal Double Hinge

– Standard Length MR4996, 33 cm

This is a lightweight and strong double hinge with a pre-set 0 degree extension stop. The hinge arms are made of A6061 aluminum and the center part from stainless steel. Teflon® washers over the rivets provide a great grease free lubrication. The hinge is black coated for more corrosion resistance. The hinge arms can be shaped individually by using bending irons.

Mediroyal Active Hinge

– Standard Length MR4997, 33 cm

This hinge has been designed for activity. It's lightweight, low profile, easy to adjust, durable and has a bilateral design. The range of motion can be set by using composite stops for extension and flexion. The complete hinge is made from A6061 aluminum. Teflon® washers provide a grease free lubrication. The hinge is black coated for more corrosion resistance. The hinge arms can be shaped individually by using bending irons.

Extension/Flexion · The light weight composite stops can easily be applied to control extension and flexion. The "E"-marked stops are for extension and "F"-marked stops for flexion. The following options are available:

Extension	Flexion
0°	45°
10°	60°
20°	75°
30°	90°
40°	

Adjustment · To set the range of motion, select the right stops from the composite chart. Start with the extension

by positioning the "E"-marked stop on the EXT side of the hinge. Open the hinge slightly and push the stop in. Then do the same for flexion by positioning the "F"-marked stop on the FLX side of the hinge. Make sure to set both hinges with the same extension and flexion stops. Try the stops before applying the hinge into the brace.

Mediroyal ROM Hinge

– Standard Length MR4998, 33 cm

– Long Length MR4999, 40 cm

The Mediroyal ROM hinge has been designed to control the knee range of motion during rehabilitation. It can be set in both extension and flexion and can also be blocked. The complete hinge is made from A6061 aluminum, the stop pins are made from CNC-milled stainless steel. Teflon® washers provide a grease free lubrication. The hinge is black coated for more corrosion resistance. The hinge arms can be shaped individually by using bending irons.

Extension/Flexion · The ROM hinge can be set in the following configurations:

Extension	Flexion	Blocked
0°	90°	0°
15°	60°	15°
30°	30°	30°
60°	15°	60°
90°	0°	90°

The hinge is easy to adjust, even when it's on the knee. No tools or parts are needed and the design is bilateral.

Adjustment · The cover has a red spring-loaded safety lock. Pull that to the side to open the cover. The hinge is marked with extension and flexion. Remove the metal pin and set in the desired location. If no flexion is needed, position the pins in the "Full range" position. Make sure that you adjust both the extension and flexion sides the same. One the setting has been done, make sure that the pins are securely positioned and close the cover. Make sure that the safety lock snaps in place before you fully load the hinge.

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Mediroyal – Functional Swedish Design

Product development is a combination of ideas, material, design and know-how. At Mediroyal we have more than 20 years of experience with developing products that provides both the patient and the therapist with enhanced functionality, every day. Swedish design is unique in many ways. Quality, performance, functionality and design are all features that make a difference. That's why we are proud to label our products with "Functional Swedish Design". Because orthoses is our passion.

MediRoyal Nordic AB · Box 7052 · SE-192 07 Sollentuna · Sweden
+46 8 506 766 00 · www.mediroyal.se

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© Copyright MediRoyal Nordic AB, Sweden · 2017 · 25233DR · www.vajer.se