Royal Putty Nova

	85 g	450 g	2,25 kg
XX-Soft · White	1111	1131	1121
X-Soft · Beige	1112	1132	1122
Soft · Yellow	1113	1133	1123
Medium · Red	1114	1134	1124
Firm · Green	1115	1135	1125
X-Firm · Blue	1116	1136	1126

Featherlite Nova

	65 cc	335 cc	1650 cc
X-Soft · White	2111	2131	2121
Soft · Yellow	2112	2132	2122
Soft Plus · Lime	2113	2133	2123
Medium · Green	2114	2134	2124
Firm · Blue	2115	2135	2125

© Copyright MediRoyal Nordic AB, Sweden · 2021 · 25952DR · www.vajer.se

GB English – Instructions

Hand Exercise Putty

General information · This product is used for individual hand exercise after instruction from your therapist. If you have allergies, check with your doctor before use.

Material content: Silicone Elastomer

Instruction · The putty is worked manually with your hands, according to the resistance selected by your therapist. Follow the recommendations from your therapist on when and how often you should use it. Wash your hands before and after use.

Precautions · The putty is designed to be used for a limited time. Constant skin exposure over 24 hours can cause skin irritation for patients with sensitive skin. Avoid contact with the eyes. Wash hands before and after use.

Never leave the putty on textile surfaces and clothes as it sticks easily. The putty can leave grease stains on sensitive surfaces. Do not leave the putty without observation near children or pets.

Caring instructions · The putty can not be cleaned. The putty should not be used on other persons than the person first fitted on. At higher temperatures, the putty will become softer in resistance. Store the putty in the container or in a plastic bag when not in use.

For more information visit www.mediroyal.se

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mediroyal

mediroyal

Mediroyal – Functional Swedish Design

Product development is a combination of ideas, material, design and know-how. At Mediroyal we have more than 20 years of experience with developing products that provides both the patient and the therapist with enhanced functionality, every day. Swedish design is unique in many ways. Quality, performance, functionality and design are all features that make a difference. That's why we are proud to label our products with "Functional Swedish Design". Because orthoses are our passion.

MediRoyal Nordic AB
Box 7052 · SE-192 07 Sollentuna · Sweden
+46 8 506 766 00 · www.mediroyal.se

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