

MEDIROYAL EPI-STRAP

MR2600



The new Mediroyal Epi-Strap is a combination of a rigid clasp and a soft strap.


It's made from a high-tech fabric and a thermoplastic core that has been welded together with high-frequency welding. The result is a flexible and functional strap with a good support. The circumference is adjustable with the hook strap that can be attached anywhere on the fabric. Inside the strap there is a silicone pad, covered with textile and attached

with hook. The pad can be positioned anywhere on the inside, but the best effect is reached when placed over the larger area of the strap. The best effect is reached when the strap is attached 3-5 cm below the arm fold, on the highest point of the muscle belly. Make sure that the strap is comfortable tightened so when you do wrist extension, you will feel a slight pressure where the pad is positioned.

INDICATIONS Lateral Epicondylitis (Tennis Elbow).

MR2600 Mediroyal Epi-Strap

Universal size, 20-36 cm. The measurement is taken 3-5 centimeters below the elbow crease.

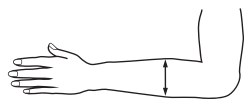

mediroyal 

Mediroyal Nordic AB
Box 7052, 192 07 Sollentuna. Sweden
Visiting address: Staffans väg 6B. Sollentuna
Tel +46 8 506 766 00 Fax: +46 8 506 766 90

www.mediroyal.se
info@mediroyal.se
facebook.com/Mediroyalnordic