



mediroyal 

Orthoses are our passion

NRX[®] STRAP KIT



NRX[®] is a registered trademark
of Mediroyal Nordic AB, Sweden
© Mediroyal Nordic AB, Sweden 2021

NRX[®] STRAP KITS

The NRX[®] Strap Kit are pre-cut applications that is ready to go within minutes. They are made from NRX[®] Strap or NRX[®] Strap PLUS and offers a wide range of adjustability and applications.



NRX1001 · NRX[®] Strap Kit for Hammertoes

A pre-cut kit for hammertoe applications. Consists of a double loop and a single loop depending on the need along with an anchor strap that goes over the foot. The loop parts will help to flex the metatarsal joint and is then secured directly on the anchor strap that goes over the foot. Hook parts for secure application are included.

Indications: Hammertoe problems

Universal size



NRX1002 · NRX[®] Strap Kit for Supination

The pre-cut kit can be used to supinate the forearm on neurological patients. It's attached over the thumb and then the spiral goes up over the elbow joint in order to control and assist radius/ulna rotation. Hook parts for secure application is included.

Indications: Assisted supination or pronation on neurological patients or post-fracture treatments.

Universal size



NRX1003 · NRX[®] Strap Kit for Hallux Valgus

This pre-cut kit is used to control abduction of the toe in Hallux Valgus patients. Realignment of the joint will not only lead to better neurological muscle function, it can also assist in reactivation of the muscular stability of the joint as well as offering off-loading of pain in the joint. The strap is attached over the big toe and individual tension is applied over the strap that is secured over the ankle. A transverse arch band is applied to secure the application. A sock can be worn over the application, making it possible to wear actively in shoes with enough space. Hook parts for secure application are included.

Indications: Hallux Valgus

Universal size



NRX1004 · NRX® Strap Kit for Thumb Abduction

This kit can be used to support and stabilize the thumb in abduction. Can be used for neurological patients to prevent or treat light adduction contractures or be used as a functional CMC-1 support. The application enables the patient to get a functional position and it's even possible to walk with a crutch wearing it. Hook parts for secure application are included.

Indications: CMC-1 joint problems in the thumb, light adduction contractures.

Universal size



NRX1005 · NRX® Strap Kit for Hip Rotation

This kit is designed to be used for patients with internal or external rotation of the hip. The pelvic strap and the thigh cuff provides attachment points for the rotation part that is then attach onto the cuff and strap. Patients with hip problems usually have a shortening of the hip adductor which requires several physical therapy sessions. The kit can be used both as a diagnostic tool to evaluate the effect on the patient as well as after the manual therapy treatment to maintain a constant dynamic stretch over the hip adductor. The external rotation strap will also provide individual pressure over the gluteus muscles which help to extend the hip joint. The hook parts on the pelvic belt and the thigh cuff are sewn on to the applications. One of the hook parts on the rotation strap is fixed in place and the other one is adjustable.

Indications: Short adductor muscles in the hip especially adductor longus and brevis, hip extension problems.

Universal size



NRX1006 · NRX® Strap Kit Epicondylitis

This kit is made from a combination of the NRX® Strap and NRX® Strap PLUS qualities for optimal function. The main body is made from NRX® Strap to conform easily and to create an anchor for the application. It has two straps to make application easy. The reinforcement part is made from NRX® Strap PLUS and can be used to provide more counterforce and support to the extensor and flexor tendons. A pad is also included that can be positioned on the NRX® Strap body, under the NRX® Strap PLUS reinforcement for more support. Both the body and the reinforcement can be customized by cutting the length of the straps.

Indications: Lateral or medial epicondylitis

Universal size



NRX® Strap Kit – Twin Bandages for fingers and Toes

Twin bandages can be useful for starting mobilization after an injury by using the adjacent finger for support. They can also be used for immobilization of fingers and toes after sprains. The NRX® Strap has a skin friction surface on the inside to prevent slipping.

The NRX® Strap Kit Twin Bandages are available in two qualities: NRX® Strap with more flexibility, and NRX® Strap PLUS with more rigidity. This allows you to increase functional stability by using the NRX® Strap PLUS quality proximal for more stability and support over the MCP and the NRX® Strap quality distally of the PIP-joint. The twin straps have a welded hook over one of the ends and can easily be cut in the other end for a custom fit.

Indications: Fracture, strain or contusion injuries to the fingers or toes, hypermobility or for preventive use in sports.

Available in:

15 mm width, NRX® Strap quality

NRX1007-06 NRX® Strap 15 mm wide · 6-pack
NRX1007-30 NRX® Strap 15 mm wide · 30-pack

15 mm width, NRX® Strap PLUS quality

NRX1008-06 NRX® Strap PLUS 15 mm wide · 6-pack
NRX1008-30 NRX® Strap PLUS 15 mm wide · 30-pack

30 mm width, NRX® Strap quality

NRX1009-06 NRX® Strap 30 mm wide · 6-pack
NRX1009-30 NRX® Strap 30 mm wide · 30-pack

Orthoses are our passion

mediroyal 

Mediroyal Nordic AB
Box 7052, Staffans väg 6B · SE-192 07 Sollentuna, Sweden
info@mediroyal.se · www.mediroyal.se