GB
English – instructions
EXTO-15 Finger Ring

General information - Read the instruction carefully. If you got an injury, you should consult with a doctor or therapist for an individual rehabilitation plan. If you suffer from poor circulation, reduced sensibility, or diabetes, be observant on pressure marks or local swelling when you use the product. If your condition gets worse when using the product, discontinue the use and contact a doctor. If the product has been fitted by a medical trained professional like occupational therapist, physical therapist, orthopedic technician, or a medical doctor, please refer to them for advice and further instructions. Follow the washing instruction on the product label. If you have material allergies, you can find the material composition on the label. For more information, visit www.mediroyal.se

Indications - The product has been designed to support and protect the DIP and PIP finger joints against hyper-extension, like swan neck deformity or hyper-mobility. It can also be used for radial- or ulnar deviation of the joints.

Contraindications - Patients with edema and circulatory problems could be a contraindication. This should be determined by the therapist or doctor. The splint should not be used to block flexion of the joint as it might cause hyper-extension.

Pre application - Use the measurement chart to select the right size. Note that the 15 degree version has a longer lever arm that the standard Multi Finger Splint to effectively control hyper-extension. This might require some adjustment and fitting the splint individually for an optimal function. One of the sides of the splint is a little wider than the other part. This part is marked with a X-sign on the outside.

Adjustment - The splint can be adjusted if needed. Use a heat-gun and spot heat the area that needs to be adjusted. The plastic composition needs a temperature between 140-160 degrees C to become adjustable. When the area is soft, shape it against an object and avoid touching it as you might burn yourself. The splint can then be cooled with water.

Application - The splint can be used different ways to provide support. See the pictures for the different applications.

- A. Hyper-extension of PIP- or DIP joints
- B. Radial- or ulnar deviation

Pay attention to pressure marks or circulatory problems if the splint is too tight. The splint should be adjusted or replaced with a larger size.

Note that the splint should not be used to block flexion as the off-set could cause hyper-extension of the joints.

Removal - Slide the splint off the finger after use.

Material content - The splint is made from polyamide and nylon.

Caring instructions and precautions - We recommend that the splint should be washed once a week. Wash with soap and water or disinfection alcohol with minimum 70%. Patients with sensitive skin, diabetes or poor circulation should check frequently for pressure marks. Patients with allergies should check the material content before use. For more information consult www.mediroyal.se

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