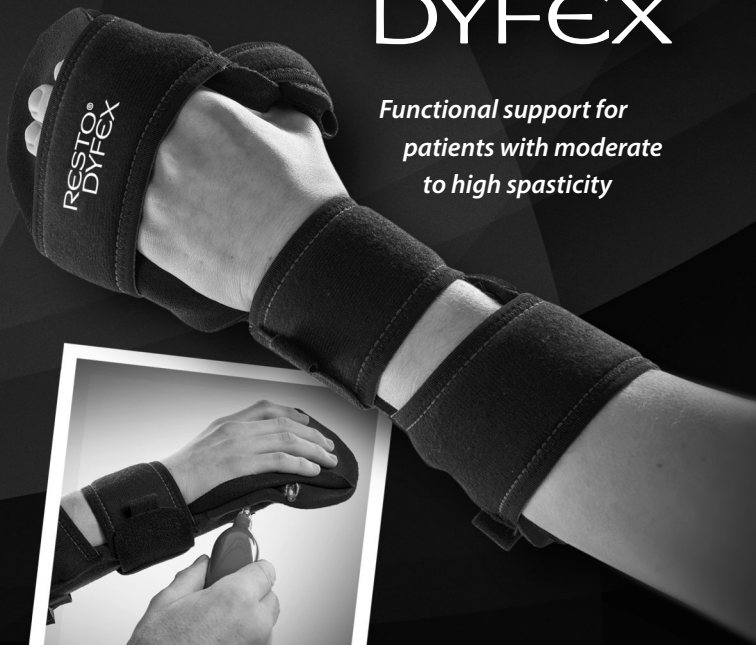
mediroyal 

RESTO[®] DYFEX

*Functional support for
patients with moderate
to high spasticity*

CE

Measurement chart

E28 RESTO DYFEX® · Available in left or right model

Size	Small	Medium	Large	
MCP width	60–75 mm	70–85 mm	85–95 mm	

Measure hand width in mm over the MCP joints


RESTO® DYFEX

*Functional support for
patients with moderate
to high spasticity*



English	10	GB
Svenska	12	SE
Suomi	14	FI
Norsk	16	NO
Dansk	19	DK
Íslenska	21	IS
Eesti	23	EE
Deutsch	25	DE
Nederlands	28	NL
Français	31	FR
Español	33	ES
Português	36	PT
Italiano	38	IT
Ελληνικά	41	EL
Český	44	CZ
Lietuvos	46	LT
Latvijas	48	LV
Slovenčina	50	SK
Slovenski	53	SI
Polski	55	PL
Română	57	RO
Русский	60	RU
Chinese Traditional	63	CH
Chinese Mandarin	66	CM
Arabic	69	AR

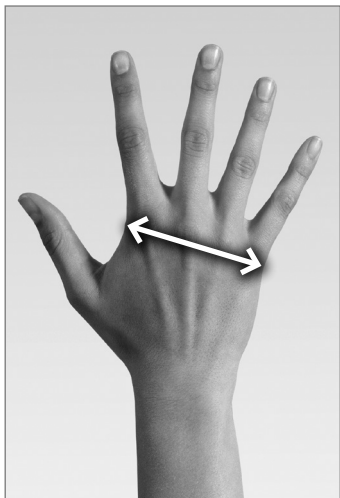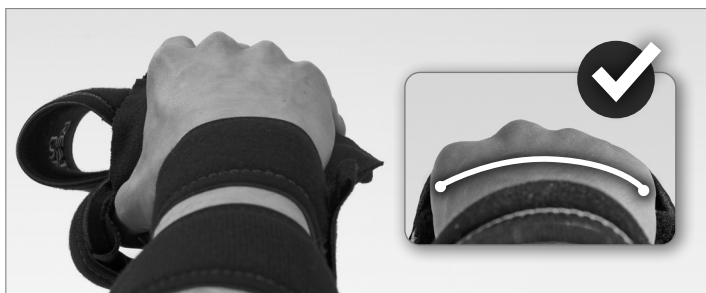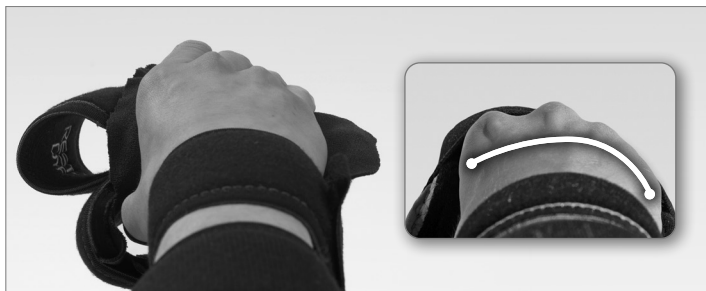Instruction

RESTO DYFEX®

General information · Read the instruction carefully. RESTO DYFEX® should always be used as a part of a rehabilitation plan decided by a doctor and therapist together. Be observant on pressure marks or local swelling when you use the product if you have poor circulation, reduced sensibility, or diabetes. If your condition gets worse when using the product, discontinue the use and contact a doctor or therapist. If the product has been fitted by a medical trained professional like occupational therapist, physical therapist, orthopedic technician, or a medical doctor, please refer to them for advice and further instructions. Follow the washing instruction on the product label. If you have material allergies, you can find the material composition on the label. For more information, visit www.mediroyal.se

Function · The RESTO DYFEX® concept has been developed and patented by Mediroyal. It consists of a malleable frame to support the lower arm, wrist, fingers, and thumb. The finger platform has two integrated inflatable bladders. The bladders complement the function of the finger platform by providing a dynamic support of the metacarpal arch and extension to the fingers. A constant and slow elongation of the flexor tendons, during a long period of time, changes the sensitivity of the muscle receptors which leads to a higher threshold for triggering spasticity.

- The first bladder is placed directly under the MCP-joints to support and give slight pressure to the metacarpal arch.
- The second bladder is placed on the finger platform, just in front of the MCP-joints. It provides support to the intrinsic muscles to extend the PIP- and DIP-joints of the fingers. We recommend that the bladders are inflated when the splint is applied and deflated when it is removed.
- The frame can be adjusted individually to a submaximal extension of the wrist and fingers. The interaction between the static frame and the dynamic bladders makes it possible to reach the individual maximum extension that is possible on your specific patient.

Material · The platform and frame are covered with CR-foam with pressure displacement function. The top cover is a microfiber that provides both high comfort and moisture transportation. This microfiber has a hookless attachment to the textile on the cover material, making it easy to apply and remove the cover without the need of hook/loop attachment. The straps are made from a stable foam material that secures the frame on the patient. The hook parts are made from low profile hook that can be attached anywhere on the outside of the frame. The cover is made from the SRX® Strap material that has a hookless attachment to the frame. It can easily be removed, and machine washed. The cover can be purchased separately as replacement.

Indications · The product has been designed to provide a functional support for patients with moderate to high spasticity. The splint also works excellently for patients that are undergoing botulinum toxin treatment for spasticity.

Contraindications · Patients with rapid fluctuations and high spasticity can be difficult to fit. For these patients we instead recommend using the TONUS® splints. Flexion contractures in the wrist should not exceed 45 degrees.

Pre-Application · To select the right size, take the width measurement over the MCP-joints then use the measurement chart.

Adjustment · The RESTO DYFEX® splint is made from a malleable aluminium frame and platform. This allows you to follow the progression of the patient by gradually adjusting it. Examine the position of the wrist, fingers, and thumb. Remove the cover from the splint before starting to adjust the frame. Adjust the frame by hand, starting from the proximal part of the frame, then over the wrist, continuing to the fingers and last adjusting the thumb. The goal should be to reach a submaximal extension without triggering spasticity. Parts of the wrist and thumb can be quite stable and might require you to bend the frame over an edge. Once you have completed the adjustment, re-attach the cover again, starting from the proximal part.

Application · Open the two straps over the platform, the finger strap, and the thumb strap. Start by applying the strap over the wrist, then the proximal strap. Close the strap over the finger platform and then adjust the thumb strap.

The splint is also fitted with a modular strap that can be used to provide more support in flexion. It can be attached on the volar part of the finger platform and run over the dorsal part of the wrist, attaching it on the proximal part of the wrist platform. It can be alternated, attaching it either radial-ulnar or ulnar-radial. See pictures in the instruction.

Adjustment of the air bladders · Once the splint has been fitted, the bladders can be adjusted. The pump included has two tips. The clear one injects air, and the black tip deflates. The valves are placed on the ulnar side of the splint. The front valve injects the finger bladder, and the rear valve injects the MCP bladder.

MCP Arch · Start by checking the position of the MCP joints before inflation. If you have an ulnar slope of the MCP III-V, increase the pressure in the rear valve to inflate the MCP bladder. Start with a small inflation, enough to support the MCP arch.

PIP/DIP Dynamic Extension · Once the MCP arch is adjusted, check the position of the fingers. The frame provides the base support and the bladders with provide the dynamic extension. Start by inflating the finger bladder gradually, carefully checking the position of the MCP and PIP/DIP. Make sure that the pressure is not hyperextending the MCP joints, in relation to the extension of the PIP and DIP-joints.

In order to prevent that, either top up the pressure in the MCP bladder or deflate the finger bladder, so you reach an optimal extension balance between MCP-PIP-DIP. The finger bladder, will once it is fully inflated, be convex in the shape to support separation of the fingers.

If you want to reach a higher degree of dynamic finger extension, deflate both bladders, adjust the finger platform into further extension and start from above with inflating both bladders.

Further adjustments · The frame is quite stable, and it is easier to bend it over a table edge. The finger platform and thumb can be adjusted by hand. Avoid bending repetitive over the same place as this might cause damages to the structure.

Time for usage · We recommend wearing the splint 1-2 hours at a time, preferably 2-3 times a day.

The splint can also be used as a night splint, but only after it has been worn for 2-3 weeks according to the above recommendation. This is to avoid that the patient develops any pressure marks from the straps or frame.

Removal of the splint · Open the straps and remove the splint. Always then check for pressure marks. We recommend deflating the bladders on removal.

Material content: Frame: Aluminium core covered with 100% CR Foam and Polyamide
Straps: 100% Polyester foam with Polyamide cover
Cover: 95% Polyester, 5% Spandex

The product is 100% latex free.

Caring instructions and precautions · We recommend that the cover is washed once a week. It can be machine washed in 60 degrees, using a laundry bag. The splint can also be machine washed in 40 degrees if needed. Patients with sensitive skin, diabetes or poor circulation should be observed for pressure marks when wearing the splint. Patients with material allergies should check the material content before use.

More information can be found on www.mediroyal.se

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Mediroyal – Functional Swedish Design

Product development is a combination of ideas, material, design and know-how. At Mediroyal we have more than 20 years of experience with developing products that provides both the patient and the therapist with enhanced functionality, very day. Swedish design is unique in many ways. Quality, performance, functionality and design are all features that make a difference. That's why we are proud to label our products with "Functional Swedish Design". Because orthoses are our passion.

Mediroyal Nordic AB

Box 7052 · Staffans väg 6 B · SE-192 07 Sollentuna · Sweden

Tel +46 8 506 766 00 · info@mediroyal.se · www.mediroyal.se