Examples of applications with SRX® Strap

FINGER

THUMB

WRIST

ANKLE

© Copyright MediRoyal Nordic AB
Sweden · 2023

26150DR · www.vajer.se

SRX® STRAP

The hookless, elastic compression strap

The SRX® Strap is based on the ARX® Strap but modified with a new lower profile textile with elastic yarn. This makes it self-attaching with a slight compression in the material. It can be cut without fraying and can be machine washed in 40 degrees C for re-use of the application.

MediRoyal Nordic AB · Box 7052 · SE-192 07 Sollentuna · Sweden
+46 8 506 766 00 · www.mediroyal.se

SRX® STRAP

GB English – Instructions

General information · Read the instruction carefully. Patients with diabetes, circulatory problems or sensitive skin should consult a doctor before use. If you suffer from allergy, check the material content before use.

Usage and function · The SRX® strap is a hookless, self-attaching strap with compression that can be applied individually over joints for increased proprioception, compression, and support. It can be cut without fraying.

Material content · 95% Polyester, 5% Elastane. The material is 100% Latex Free.

Caring instructions and precautions · We recommend to machine wash the strap once a week in 40 degrees, preferably using a laundry bag. Do not use fabric softener as the self-attaching function will then be lost. Patients with sensitive skin, diabetes or poor circulation should consult a doctor before use. Patients with allergies should check the material content before use. For more information consult www.mediroyal.se.

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