GB English – instructions
Universal Wrist UNI801 · UNI802

General information · Read the instruction carefully. If you got an injury, you should consult with a doctor or therapist for an individual rehabilitation plan. If you suffer from poor circulation, reduced sensibility, or diabetes, be observant on pressure marks or local swelling when you use the product. If your condition gets worse when using the product, discontinue the use and contact a doctor. If the product has been fitted by a medical trained professional like occupational therapist, physical therapist, orthopedic technician or a medical doctor, please refer to them for advice and further instructions. Follow the washing instruction on the product label. If you have material allergies you can find the material composition on the label. For more information, visit www.mediroyal.se

Indications · The product has been designed to provide volar support to the wrist.

Contraindications · Patients with instable fractures should not use this brace as an alternative for a cast.

Pre Application · To select the right size, take the circumferential measurement around the wrist. Use the measurement chart to determine the correct size. Note that the product is made bilateral.

Adjustment · The brace is made bilateral, however the splint is mounted as right in default. To switch the product into a left, remove the splint from the pocket and insert it in the other opposite pocket. Make sure that the bending of the splint fits the palm. The thumb part fixed and the thumb hole can be cut larger if needed. Do not cut over the upper zick-zag seam.

Application · Open the hook straps and slide the brace on the hand. Attach the hook parts to the body. They can be attached anywhere on the main body. Note that the closure will be different on right and left side, as the design is asymmetrical.

Removal · Open all the hook parts and slide off the brace. Close all straps when the brace is not in use.

Material content · Check the care label for the material composition if you have material allergies.

Caring instructions and precautions · We recommend that the brace should be washed once a week. Follow the washing instructions on the care label, close all straps and use a laundry bag. The splints should be removed. Patients with sensitive skin, diabetes or poor circulation should check frequently for pressure marks. Patients with allergies should check the material content on the care label before use. For more information consult www.mediroyal.se

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