English – instructions

Finger Extension Springs · 501, 502, 504, 507, 508, 602

General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content under the point "Material content".

Indications · The product has been designed to provide extension force to a contracted finger. The general philosophy is to apply a low tension over a long period of time to allow extension of the contracture. This means that the treatment usually has to cover several weeks. Check with your therapist for details. The product has been designed for flexion contractures, extension deficiency of the PIP joint, post fracture treatment and Boutonniere deformity.

Contraindications · Patients with severe inflammation processes in the PIP joint, patients with excessive edema, patients with poor circulation, patients with reduced sensitivity or patients with stiff contractures should consult with a physician before use.

Pre Application · To select the right size, use the measurement chart in this instruction. It's also important to select the right tension depending on the need of the patient. Always start with a low tension.

502 – Extra High tension 12,4 N at full extension

507 – High tension 3,4 N at 15 degrees of flexion

508 – High tension 3,4 N at 15 degrees of flexion

501 – Medium tension 2,3 N at 15 degrees of flexion

504 – Medium tension 2,3 N at 15 degrees of flexion

602 – Low tension 1,1 N at 15 degrees of flexion

Adjustment · 501, 504, 507, 508 can all be adjusted in extension force. To increase the extension force, press the pads a part for a few seconds and hold. Then release and try it of the finger. To decrease, press the two pads together and hold for a few seconds. Then try on the finger. 502, 602 Can't be adjusted.

Application · Apply the spring over the finger. Make sure that it's attached well to the base and covers the joints that are affected. The 502 has a Velcro strap that is attached to the hook part.

Removal of the splint · Simply slide it off the finger.

Material content · The finger splints are made from stainless steel, aluminium and PE-foam. All products are latex free.

Caring instructions and precautions · We recommend that the cover should be washed once a week. The PE-foam is non-absorbent and can be disinfected by using alcohol or a mild detergent. Do not use a detergent containing bleach. Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content under point 8. For more information consult www.mediroyal.se

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Finger Extension Springs Size chart


501 · Slightly behind the MCP joint to the crease of the DIP joint					
Size	XS (AA)	S (A)	M (B)	L (C)	XL (D)
Length	51 mm	64 mm	76 mm	89 mm	102 mm
Width	22 mm	22 mm	25 mm	25 mm	29 mm

502 · Slightly behind the MCP joint to the crease of the DIP joint			
Size	S (A)	M (B)	L (C)
Length	64 mm	70 mm	76 mm
Width	22 mm	22 mm	25 mm

504 · In front of the MCP joint to the crease of the DIP joint				
Size	XS (AA)	S (A)	M (B)	L (C)
Length	38 mm	44 mm	51 mm	57 mm
Width	22 mm	22 mm	25 mm	29 mm

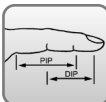507 · Slightly behind the MCP joint to the crease of the DIP joint				
Size	XS (AA)	S (A)	M (B)	L (C)
Length	51 mm	64 mm	76 mm	86 mm
Width	22 mm	22 mm	25 mm	29 mm


508 · In front of the MCP joint to the crease of the DIP joint				
Size	XS (AA)	S (A)	M (B)	L (C)
Length	38 mm	44 mm	51 mm	57 mm
Width	22 mm	22 mm	25 mm	29 mm

602 · In front of the MCP joint to the crease of the DIP joint. Can also be used on the DIP joint.

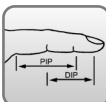Size	XS (AA)	S (A)	M (B)	L (C)	XL (D)
Length	38 mm	44 mm	51 mm	57 mm	67 mm
Width	22 mm	22 mm	25 mm	29 mm	29 mm

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© Copyright MediRoyal Nordic AB · Sweden · 2026 · 26354DR · www.vajer.se