MR2703 - Proxi® MP-splint				
Size	S	M	L	
mm	35–50	50–60	60–	

Available in left and right versions

The measurements refer to the length of the metacarpal bone.

© Copyright MediRoyal Nordic AB - Sweden - 2026 - 26360DR - www.vajer.se

English – instructions

Proxi MP

General Information - Read the instruction carefully. If you got an injury, you should consult with a doctor or therapist for an individual rehabilitation plan. If you suffer from poor circulation, reduced sensibility, or diabetes, be observant on pressure marks or local swelling when you use the product. If your condition gets worse when using the product, discontinue the use and contact a doctor. If the product has been fitted by a medical trained professional like occupational therapist, physical therapist, orthopedic technician or a medical doctor, please refer to them for advice and further instructions. Follow the washing instruction on the product label. If you have material allergies you can find the material composition on the label. For more information, visit www.mediroyal.se

Indications - The product is designed to provide support and relief to the thumb metacarpophalangeal (MP) joint.

Contraindications - The product should not be used without prior assessment by a therapist or physician.

Pre application - Use the measurement chart to select the right size. Note that the length of the platform is different for each size. If you are in between, choose the smaller size.

Adjustment - The splint is made from a molecular modified thermoplastic that has been reinforced with a metal splint that can be adjusted individually. Make sure that the platform conforms to the thumb with a good fit. The splint can be adjusted if needed by using hot water, 75-80 degrees C. Start by adjusting the metal splint in flexion, then immerse the complete splint in the hot water. When soft, remove it and place it on a dry towel to absorb the moisture. Then fit the splint on the thumb and adjust the form. Before fitting, dry the inside with paper towel.

Application - Close the thumb strap first. Pull the strap around the wrist and secure it with the hook piece. The length of the strap can be adjusted.

Apply the banana shaped double hook onto the volar side of the splint, over the thenar eminence. Then apply the coin shaped double hook on the top over the attachment of the second strap. Both hook parts should have the black side facing down to prevent the hook from detaching when opening the strap. Pull the second strap from the attachment point and attach onto the banana-shaped hook, then adjust the tension in the strap and attach to the coin hook.

When correctly applied, the MP-joint should be immobilized, and flexion of the IP-joint can be restricted. If more flexion is needed over the IP-joint, the splint can be adjusted.

The angle of the MP joint can be adjusted in flexion, to position the thumb in a comfortable resting position if needed.

Removal - Open the straps and remove. Close all straps when not in use.

Material content - If you suffer from allergies, check the care label for the material composition.

Caring instructions and precautions - We recommend that the splint should be washed once a week. Follow the washing instructions on the care label, close all straps and use a laundry bag. Do not exceed the maximum temperature recommendation of 40 degrees Celsius as the thermoplastic might be deformed. Patients with sensitive skin, diabetes or poor circulation should check frequently for pressure marks. Patients with allergies should check the material content on the care label before use. For more information consult www.mediroyal.se

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