
Size chart

Size	XS	S	M	L	XL
cm	13–14	14–16	16–18	18–20	20–22
		Left	Right		

© Copyright MediRoyal Nordic AB, Sweden · 2021 · 25814DR · www.vajer.se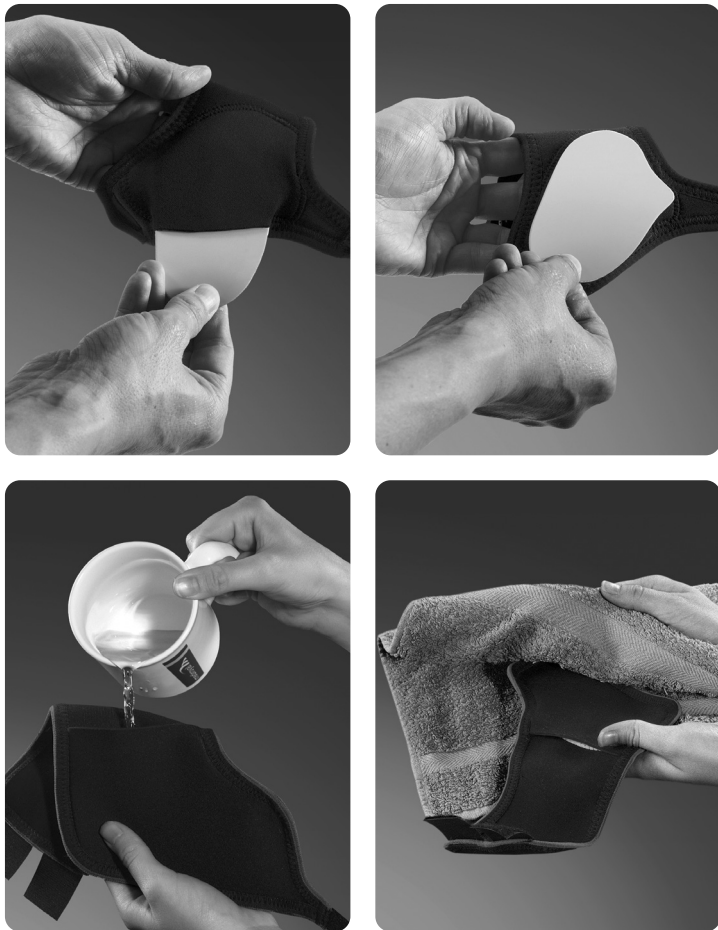

English – instructions

Thenax Opponent Thumb

General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the backside of the white textile label.

Indications · The product has been designed to provide support to the CMC-joint.

Contraindications · Patients with instable fractures should not use this brace.

Pre Application · To select the right size, take the circumferential measurement around the wrist. Use the measurement chart to determine the most suitable size. Note that the product made in a left or right version.

Skin application: If fitted on the skin directly, heat the material in hot water 75-80 degrees C. It takes usually 1-2 min before its soft. Dry the surface on a towel and then apply onto the patient. When the material is cool, insert it into the pocket.

Application · Heating inside the pocket: Place the thermoplastic inside the pocket. Fold the material slightly to open up the pocket. Use hot water 75-80 degrees C and pour it down into the pocket. Be careful not to burn your fingertips. Once the pocket is full let the orthosis stay for about 1 minute. Check that all parts of the thermoplastic are soft, if not pour more water into the pocket. When soft, pour out the water and place the orthosis on a dry towel. Fold and press to absorb moisture. Always check the temperature of the orthosis before skin application. Patients with sensitive skin should wear a protective sleeve.

Adjustment · The elastic hook strap will provide individual support to the CMC-joint and will create additional joint space. The strap can then be pulled tighter to the attachment just under the CMC-joint to provide more support. The remaining part of the strap can be attached on to the dorsal side.

Removal · Open all the straps and remove the brace. Close all straps when the brace is not in use.

Material content · Check the material content on the white label.

Caring instructions and precautions · We recommend that the brace should be washed once a week. Follow the washing instructions on the white label on the product, close all straps and use a laundry bag. Never wash in temperature over 40 degrees Celsius.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the backside of the label on the cover.

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MediRoyal Nordic AB · Box 7052 · SE-192 07 Sollentuna · Sweden