English – instructions
C-GO® Thumb

General information - Read the instruction carefully. If you got an injury, you should consult with a doctor or therapist for an individual rehabilitation plan. If you suffer from poor circulation, reduced sensibility, or diabetes, be observant on pressure marks or local swelling when you use the product. If your condition gets worse when using the product, discontinue the use and contact a doctor. If the product has been fitted by a medical trained professional like occupational therapist, physical therapist, orthopedic technician, or a medical doctor, please refer to them for advice and further instructions. Follow the washing instruction on the product label. If you have material allergies, you can find the material composition on the label. For more information, visit www.mediroyal.se

Indications - The product has been designed to stabilize, support and unload the CMC- and MP-joint of the thumb

Contraindications - Patients with edema, diabetic wounds or circulatory problems should not use this product unless advised by a therapist or doctor.

Pre application - To select the right size, take the circumference measurement around the wrist. Use the measurement chart to select the right size.

Application - Open all the hook straps and apply over the thumb. Close the upper hook strap first, then attach the lower one. Adjust the fit gradually so the main body fits well. Use the center strap to support the thumb individually. The strap has an integrated hook on the inside that can attach to the surface on the main body. The hook location is where the gray lamination is positioned on the strap. Stretch the strap and apply it over the thumb to provide individual abduction and thenar eminence support. Attach the hook part on the dorsal side. Adjust the compression and support if needed.

The thumb part can be trimmed down if the IP-joint needs more flexion. Simply cut with a pair of scissors in the material but do not cut the dorsal seam.

Removal - Open all the hook straps and remove the orthosis. Close the hook straps when the orthosis is not in use.

Material content - If you suffer from allergies, check the care label for the material composition. It can be wise to start using the product for a shorter time and then check the skin condition before using it for a longer time.

Caring instructions and precautions - We recommend that the brace should be washed once a week. Follow the washing instructions on the care label, close all straps and use a laundry bag. Patients with sensitive skin, diabetes or poor circulation should check frequently for pressure marks. Patients with allergies should check the material content on the care label before use. For more information consult www.mediroyal.se

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© Copyright Mediroyal Nordic AB, Sweden · 2026