GB

English – instructions

Thermoplastic Thumbs

General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on white textile label.

Indications · The product has been design to provide support to the CMC- and MP-joints of the thumb after trauma, post-operative or when immobilization is needed.

Contraindications · Patients with instable fractures should not use this brace.

Pre Application · To select the right size, take the circumferential measurement around the wrist. Use the measurement chart to determine the most suitable size. Note that the product made in left or right versions.

Adjustment · The brace is fitted with a thermoplastic piece that should be moulded individually to the patient. There are two ways to do that.

Moulding inside the pocket: This is the easiest way to mould it. Insert the thermoplastic piece flat into the pocket. Fold the plastic in the center of the opening, creating a fold that holds the opening. This will make it easier to pour the hot water into the pocket. Use water that is 75-80 degrees C and carefully pour it into the pocket. Be careful not to burn yourself. Let the water stay at least 60 seconds in the pocket. Check that the plastic insert is soft in all areas. If not, simply refill the pocket with more water. Once soft, pour out the water and place the brace flat on a dry towel. Fold the towel carefully to absorb the moisture. Close the thumb grip first and slide the brace onto the thumb. Stretch slightly over the CMC joint when attaching the strap, this will prevent wrinkles on the inside. After approx 3-4 minutes the thermoplastic part starts to become stable. You can then remove the brace and let it cool. Allow at least 10 minutes cooling before starting to load the brace. Use a dry paper towel to remove the last moisture of the edging tapes. If re-moulding is necessary simply open up the pocket again when the plastic is cool, and pour hot water in it again.

Moulding outside the pocket: Try the brace on to determine that you have the right size. Mark the position of the brace on the patient using a skin marker. Take the thermoplastic piece and immerse into hot water 75-80 degrees C. The material turns soft in approx 60 second. Remove it from the water and dry it quickly on a towel. Apply the thermoplastic piece to the thumb and let it cool. It usually takes less than 1 minute. You can apply it into the pocket once the shape is stable which takes 4-5 minutes usually. If further adjustment is needed simply immerse the plastic into the water again and start all over.

Removal · Open the straps and remove the brace. Close all straps when the brace is not in use.

Material content · Check the white label on the product for material content.

Caring instructions and precautions · We recommend that the brace should be washed once a week. Follow the washing instructions on the label on the product, close all straps and use a laundry bag.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the label.

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© Copyright MediRoyal Nordic AB, Sweden · 2016 · 250550R · [www.vajer.se](#)