GB**English – instructions**

Thumb Supports

General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the backside of the white textile label.

Indications · The product has been designed to provide support to the CMC- & MP-joint of the thumb.

Contraindications · Patients with instable fractures should not use this brace.

Pre Application · To select the right size, take the circumferential measurement around the wrist. Use the measurement chart to determine the most suitable size. Note that the product made in left or right versions.

Adjustment · The brace comes with two splints; one spiral stay that is more flexible and a coated steel splint that provides more stability. Your therapist will decide which one is more appropriate for you. The brace is fitted with a splint which can be removed through the opening in the end of the pocket. The splint can be adjusted to the patient to prevent pressure points.

Application · Close the thumb grip of the brace before application. Pull the thumb through the opening. Apply the strap around the wrist and re-adjust the thumb strap if needed.

Removal · Open the Velcro straps and remove the brace. Close all straps when the brace is not in use.

Material content · Check the backside of the white label for material content.

Caring instructions and precautions · We recommend that the brace should be washed once a week. Follow the washing instructions on the white label on the product, close all straps and use a laundry bag.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the backside of the label.

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© Copyright MediRoyal Nordic AB, Sweden · 2021 · 25813DR · www.vajer.se
MediRoyal Nordic AB · Box 7052 · SE-192 07 Sollentuna · Sweden