GB

English – Instructions

Knee Support CTX – General

General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the textile label.

CTX is a line of compression textile braces for the knee, ankle and elbow. The compression is graded with three compression zones. The higher pressure is located around the joint then grading off into two lower compression zones. This helps to remove swelling over the joint when you are active. Swelling can otherwise cause problems with range of motion in the joint and lead to pain.

Patients with circulatory problems or cardiovascular disease should always check with their physician before use. Only use the support when you are active, never during the sleep.

Indications · The product has been design to provide support and compression to general knee injuries like mild instabilities, local swelling after trauma, inflammation due to over-loading, poor joint position sense or osteoarthritis.

Contraindications · Patients who haven't received a correct diagnosis from a doctor should consult one before use of the brace.

Pre Application · To select the right size, take the circumference measuement around the knee at the height of the knee cap. Use the measurement chart to determine the most suitable size. Open all Velcro straps of the brace before application.

Application · Apply the brace by either pulling it up on the leg. Make sure that the knee cap is placed in the center hole in the front. Close the Velcro straps.

Removal · Open the Velcro straps and remove the brace. Close all straps when the brace is not in use.

Material content · Check the label for material content.

Caring instructions and precautions · We recommend that the brace should be washed once a week. Follow the washing instructions on the white label on the product, close all straps and use a laundry bag. If the brace has spiral stays they don't have to be removed.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the backside of the label on the cover.

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© Copyright MediRoyal Nordic AB, Sweden · 2016 · 25086DR · www.vajer.se