## English – instructions

EXTO™ Light – Extension Orthosis for Posture

**General information** - Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the white textile label.

**Indications** - The product has been designed to provide a posture support and extension assistance for patients with poor posture. It's also excellent to use for patients suffering from osteoporosis. Osteoporosis can depend on several factors like age, smoking, physical activity, oral medication with cortisone and the risk of falling. It's important that your doctor evaluate the different treatments needed. Physical activity is an important factor for treatment of osteoporosis. EXTO™ Light can help you to perform the activities you want.

**Contraindications** - Patients with recent and instable fractures should use another treatment method.

**Pre Application** - To select the right size, take the circumferential measurement around the waist. Use the measurement chart to determine the most suitable size. The brace is adjustable and the size depends sometimes on the tolerance for pressure by the individual patient.

**Adjustment** - The brace is made of a breathable and comfortable material. It has a function design with several different material areas to provide the best wearing comfort and performance for the patient. The straps that attaches on the side of the brace are constructed to provide the individual extension support. The nylon straps provide high friction which makes it easy to pull the straps, even with low hand strength.

**Application** - Open the straps on the side and then the waist belt of the brace. There are tabs to make opening and closing easier. If the tabs are not needed, they can be cut away. Adjust the length of the arm straps. Apply the brace like a back pack, slide first one arm into the loop, then grab the other and slide in the other arm. Make sure that the straps over the shoulders have the fleece material on the inside and that the straps are not twisted. Secure the waist band over the lower part of the waist. There are silicone strips in the back of the brace to provide suspension and to prevent it from sliding up. Make sure that the brace is securely applied and that the pressure doesn't disturb breathing. Take the straps on the side and pull them down and out to provide extension to the upper part of the body. The combination of the different materials provides a dynamic extension that can be tolerated by most patients without extensive pressure over the shoulders. Wear the brace for 5 minutes, then adjust the straps to get more extension if needed. A successful orthotic treatment for osteoporosis is usually a combination of several orthotic options in combination with physical therapy. EXTO™ Light can serve as a first step to provide extension and posture support for the patient.

**Removal** - Open the straps on the side and attach them further back to slack the compression in the shoulder straps. Then open the waist part. Slide one arm out then the other one. Re-attach the hook parts, that increase the life expectancy of the brace.

**Caring instructions and precautions** - We recommend that the brace should be washed once a week. Follow the washing instructions on the white label, use liquid detergent, close the straps and always use a laundry bag.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on label on the product.

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