GB **English – instructions**
Omox Arm Sling

General information

Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the textile label.

Indications

The product has been designed to provide support and immobilization of the shoulder and elbow after soft tissue injuries.

Contraindications

Patients with instable fractures should not use this product unless the fracture has been stabilized.

Pre Application

Select the right size using the sizing reference on the measurement chart.

Adjustment

The length of the pocket is important for the support of the sling. Make sure to select the right size, rather a little too long than too short. The strap around neck is adjustable by the hook strap. If your product is fitted with a belt, it's also adjustable by the hook straps.

Application

Open the hook strap and slide the arm into the pocket. Adjust and secure the hook strap around the neck. If the product is fitted with a waist band adjust it and secure the hook. The product has been fitted with a thumb loop to provide additional support.

Removal

Open the straps and remove the sling.

Caring instructions and precautions

We recommend that the brace should be washed once a week. Follow the washing instructions on the white label on the product, close all straps and use a laundry bag.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the label.

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© Copyright MediRoyal Nordic AB, Sweden · 2016 · 25076DR · www.vajer.se