MR328 DYFEX® FHP

Functional Hand Position

MR329 DYFEX® RHP

Resting Hand Position

DYFEX® is a registered trademark
of Mediroyal Nordic AB, Sweden
Patents pending

English · 9–11

GB

Svenska · 12–14

SE

Suomi · 15–17

FI

Deutsch · 18–20

DE

Français · 21–23

FR

Italiano · 24–26

IT

Español · 27–29

ES

Português · 30–32

PT

Nederlands · 33–35

NL

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EL

Eesti · 40–42

ET

Latvijas · 43–45

LT

Lietuvos · 46–48

LV

Polski · 49–51

PL

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RU

Češka · 55–57

CZ

Arabic · 58–60

AR

Persian · 61–63

PR

Chinese · 64–65



CH

Traditional Chinese · 66–67

TW

Japanese · 68–70


JP

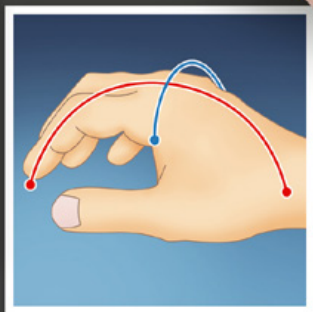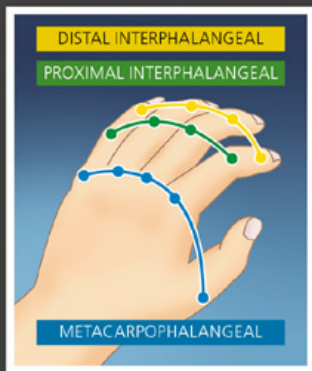A

B

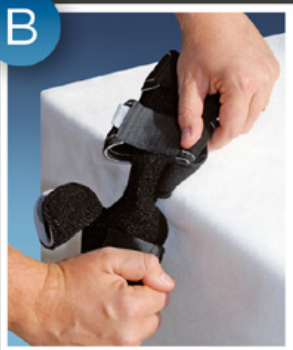B

B

C

C

C

D

D

D

D

D


D

D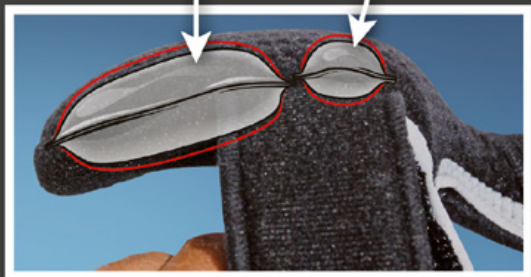

English – instructions

328 Mediroyal DYFEX® FHP – Functional Hand Position

329 Mediroyal DYFEX® RHP – Resting Hand Position

General information

Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the white label.

Function

The DYFEX® concept has been developed and patented by Mediroyal. It consists of a malleable frame to support the lower arm, wrist, fingers and thumb in conjunction with two inflatable bladders. These bladders are integrated on the finger platform and work together to provide support to the transversal and longitudinal arches. The bladders complement the function of the finger platform by providing a dynamic extension of the fingers. A constant and slow elongation of the flexor tendons, changes the sensitivity of the receptors. That way the threshold for triggering spasticity will be higher.

- The first bladder is placed directly under the MCP-joints to support and give slight pressure to the metacarpal arch. The second bladder is placed on the finger platform, just in front of the MCP-joints.

It provides support to the intrinsic muscles to extend the PIP- and DIP-joints of the fingers.

- The frame should be adjusted individually, to reach a submaximal extension of the wrist and fingers. The interaction between the static frame and the dynamic bladders makes it possible to reach the individual maximum extension that is possible on your specific patient.

Material

The material is made from pressure displacing and ventilated foam. The top surface has a textile material with a check-board pattern that has been specially developed for the DYFEX® products. The material works as an active pressure displacement function, wicks away moisture from the skin and provides an optimal air-flow to the skin area. The open check-board pattern makes it easy to keep the material clean.

Indications

The product has been design to provide a functional support to patients with moderate to high spasticity. It also works excellent for patients that are undergoing botulinum toxin treatment. Volar contractures in the wrist should not be over 45 degrees.

Contraindications

Patients high spasticity can be difficult to fit if they have a rapid fluctuations in spasticity. Patients with volar contractures over 45 degrees in the wrist.

Pre Application

To select the right size, take the width measurement over the MCP-joints then use the measurement chart **(Pictures A)**.

Adjustment

The DYFEX® splint is made from a malleable aluminium frame and platform. This will allow you to gradually adjust the splint depending on the progress of the patient. Carefully evaluate the wrist, fingers and thumb. Adjust the frame by using your hands, to reach a submaximal range. Start from the wrist, then adjusting the finger platform and last the thumb. The wrist area can be quite stable. It might be easier to bend the splint over an edge. The thumb piece on the 328 MediRoyal DYFEX® FHP can't be placed into abduction. If you need abduction of the thumb, use the 329 MediRoyal DYFEX® RHP instead **(pictures B)**.

Application

Open the Velcro straps. Position the fingers, thumb and wrist into the splint. Start by closing the straps around the wrist to secure the brace. Then attach the strap over the fingers. The strap that runs from the ulnar side should be attached to the radial side between the thumb and index finger. It's important to use this strap to provide more support from the frame and to position the patient correctly over the MCP-bladder. Make sure that the straps are not too tight as that might develop edema **(Pictures C)**.

Adjustment of the air bladders

Once the patient fits into the frame, the bladders can be adjusted. The pump that has been included has two tips. The clear one injects air and the black tip deflates. Insert the clear ending into the lower valve, on the lower part of the outside of the splint. Let the patient rest in the frame. Before pumping, check the position of the MCP-joints. Inflate once and carefully follow the position of the MCP-joints. It usually requires only one or two pump strokes to achieve a good support.

After the first initial pumping of the MCP-bladder, turn to the finger bladder. The valve is placed on the ulnar side of the finger platform. Insert the clear tip of the pump. Inflate with two full pump strokes to achieve finger extension and support of the PIP and DIP arches. When you have reached a desired finger extension, move back to the MCP-bladder. Insert the clear tip of the pump in the valve on the lower part of the outside again. Top up the MCP-bladder with another pump stroke. It's important that the dynamic extension of the fingers is supported correctly from the MCP-joints. Once the MCP-bladder has been topped up, you might have to top up the finger bladder again.

When the finger bladder is full, the shape will be convex. This will assist the finger position as well as supporting the PIP and DIP arches. If you want to reach a higher degree of dynamic finger extension, deflate both bladders, adjust the finger platform into further extension and start from above with inflating the bladders **(Pictures D)**.

Further adjustments

The frame is quite stable and it's easier to bend it over a table edge. The finger platform and thumb can be adjusted by hand. Avoid bending repetitive over the same place as this might cause damages to the structure **(Pictures E)**.

Time for usage

The best effect is reached if the splint is applied for 2 hours a couple of times per day. Try to spread the usage over the complete day. The splint can also be used as a night splint, but only after it has been worn for 2-3 weeks according to the above recommendation. This is to avoid that the patient develops any pressure marks from the straps or frame.

Removal of the splint

Open the Velcro straps and remove the splint. Always then check for pressure marks.

Caring instructions and precautions

We recommend that the splint should be washed once a week. Follow the washing instructions on the white label on the cover and always use the laundry bag.

Patients with sensitive skin, diabetes or poor circulation should be observed. Patients with allergies should check the material content on the label.

More information can be found on www.mediroyal.se

DYFEX® is a registered trademark of MediRoyal Nordic AB, Sweden.
Patents Pending.

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MR328 DYFEX® FHP

Functional Hand Position

DYFEX® stands for Dynamic Finger Extension and is a new concept, which has been developed and patented by Mediroyal, for patients with spasticity. The concept is a combination of a static but adjustable orthosis that has been fitted with two air bladders. The bladders complement the function of the finger platform by providing a dynamic extension of the fingers.

DYFEX® is a registered trademark of Mediroyal Nordic AB.
Patents pending.

MediRoyal Nordic AB
Box 7052 · S-192 07 Sollentuna · Sweden
+46 8 506 766 00 · www.mediroyal.se