

## NRX® STRAP

- One strap, unlimited options!

## **NRX® Patella Tendon**

Material

NRX® Strap 50 mm and NRX® Strap PLUS in 30 mm or NRX® Strap 30 mm



Cut the 50 mm strap so the length covers the circumference of the knee plus 3 cm. Attach the hook tab to one side. Cut the reinforcement strap from NRX® Strap PLUS 30 mm for more static support or from NRX® Strap 30 mm for more dynamic support. Make sure that the length is at least 4–5 cm.



Attach the strap around, covering the patella tendon, tibial tuberosity and apex patella.





Attach the reinforcement strap in the center of the patella tendon. Stretch the strap and apply the force evenly over the tendon.

## **Caring instructions**

The NRX® strap can be washed in 40 degrees machine wash with a liquid detergent. Always use a laundry bag. Remove the hook parts before washing and let air dry.

© Mediroyal Nordic AB 2019