Mediroyal Nordic AB · Box 7052 · Staffans väg 6 B · SE-192 07 Sollentuna · Sweden
Tel +46 8 506 766 00 · info@mediroyal.se · www.mediroyal.se

© Copyright MediRoyal Nordic AB, Sweden · 2019 · 25521DR · www.vajer.se

TFCC Wrist Orthosis

TFC101 · TFC102 · TFC103

English

GB

Svenska

SE

Boa® is a registered trademark of BoaTechnology Inc.

GB

English · Instructions

Mediroyal TFCC Wrist Orthosis

This wrist orthosis has been designed to immobilize and control pronation and supination of the wrist after injuries to the TFCC ligaments. In order to achieve effective rotation stability, the orthosis length must be at least 2/3 of the length of the forearm. Therefore, the model is available in three individual lengths:

- TFC101 Mediroyal TFCC Wrist Orthosis, 25 cm length in sizes XS-XL, Left & Right
- TFC102 Mediroyal TFCC Wrist Orthosis, 28 cm length in sizes XS-XL, Left & Right
- TFC103 Mediroyal TFCC Wrist Orthosis, 31 cm length in sizes XS-XL, Left & Right

General information · Read the instruction carefully. The product should be fitted by a medical professional like an occupational therapist, physical therapist, orthopedic technician or a medical doctor. Patients with diabetes, circulatory problems or sensitive skin should consult their doctor before use. If the patient is allergic, check the material content on the textile label on the product.

Indications · This wrist orthosis has been developed to support the wrist after operation of the TFCC ligaments but also for conservative treatment. The orthosis provides stability and rotation control. It can also be used after early cast removal of general wrist fractures to immobilize the wrist joint.

Contraindications · If you are suffering from reduced sensation or circulatory problems you should consult your medical practitioner before use. If you experience impaired sensation, numbness or circulatory problems in the wrist or fingers, release the compression and contact your medical practitioner.

Application and adjustment · Note that the wrist should be held in neutral position with the elbow in approx. 90 degrees during application. See the instruction pictures for more information.

- Open the Boa® system by lifting the dial until you hear a click. Then open the three straps. Observe the different numbers on the straps and the attachment points on the upper side of the orthosis.
- Pull up the orthosis over the wrist without changing the neutral position of the wrist and forearm.
- Apply the middle strap, marked with number 1 first. Make sure that it's attached over the matching number 1 on the upper part.
- Continue with the lower strap marked with number 2 and attach it over the matching number 2 on the upper part.
- The last strap, number 3 should be attached as close to the edge of the upper part of the orthosis.

Engage the Boa® system by pressing on the dial. Then turn the dial carefully until the stability feels comfortable. Test the stability for a few minutes and adjust the tension if needed. If you get loss of sensation

or increased pain, lift the dial and pull the plastic grip to release compression. Start from the beginning and add less compression. The rotation stability is based on a combination of length of the orthosis and the compression from the Boa® system. It's important to find the optimal individual balance for you without too high compression.

Time usage · After the surgery the orthosis should be worn both night- and daytime or according to the specific instructions from your medical practitioner. It's also your medical practitioner that decides for how long you will wear the orthosis. The orthosis should be removed on a daily basis for ventilation of the skin and hygiene according to the instructions provided by your medical practitioner. This is important to prevent skin- or pressure problems.

Additional rotation control · If further rotation control is needed, the TFCC Elbow Extension Unit (TFC105) can be used to control the elbow joint and increase the lever. See the separate instruction for more information.

Biofeedback-Proprioception · The inside of the orthosis has friction surfaces that actively help to signal movement and rotation. These surfaces can in rare cases cause skin irritation. If this would occur we recommend to use an arm sleeve to protect the skin.

Arm Sleeves · The arm sleeves can both protect the skin as well as provide compression over the wrist to prevent and treat swelling of the wrist that could occur during long term immobilization. The arm sleeve has compression class 2 over the wrist. The sleeve has a seam that should be positioned on the outside in order to prevent pressure.

The arm sleeves are available in two lengths:

- TFC107 Short Arm Sleeve S-M-L (that fits TFC101)
- TFC108 Long Arm Sleeve S-M-L (that fits TFC102 and TFC103).

Material content · The material content of the product is described on the white care label on the product.

Caring instructions and precautions · We recommend that the orthosis should be washed once a week. Follow the washing instructions on the care label of the product. Remove the metal splint, close the straps and lock the Boa® dial by pressing it down. Use a laundry bag, liquid detergent and wash in a washing machine but do not tumble.

Patients with sensitive skin, diabetes or poor circulation should be observed for pressure marks or swelling. Patients with allergies should observe the material content. Should discomfort, skin irritation, increased pain or reduced sensation occur during your treatment, immediately contact your medical practitioner.

For more information visit www.mediroyal.se

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