

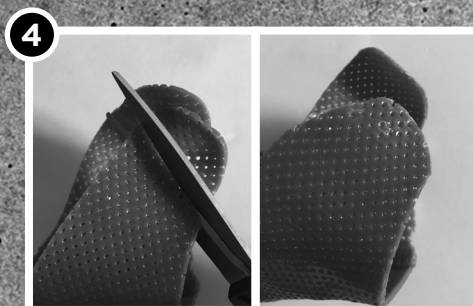
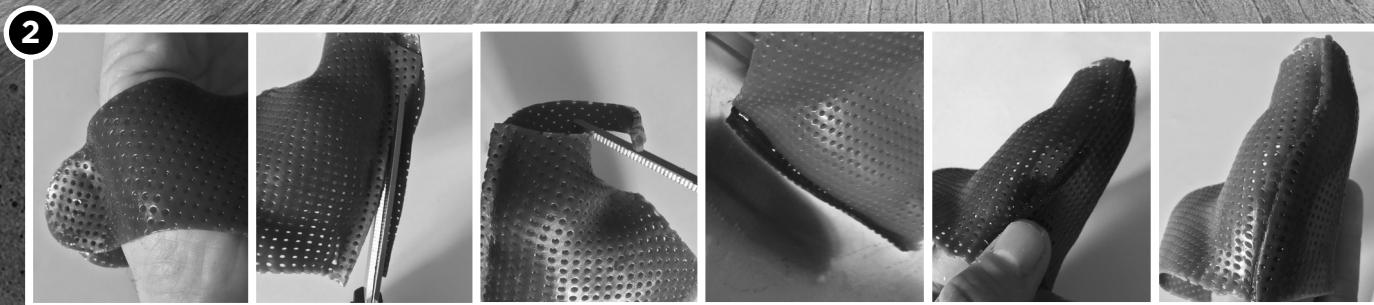
mediroyal

Orthoses is our passion

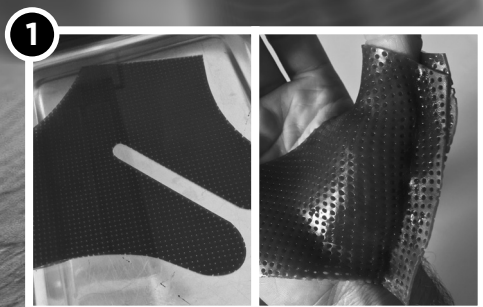Instruction Thumb Gauntlet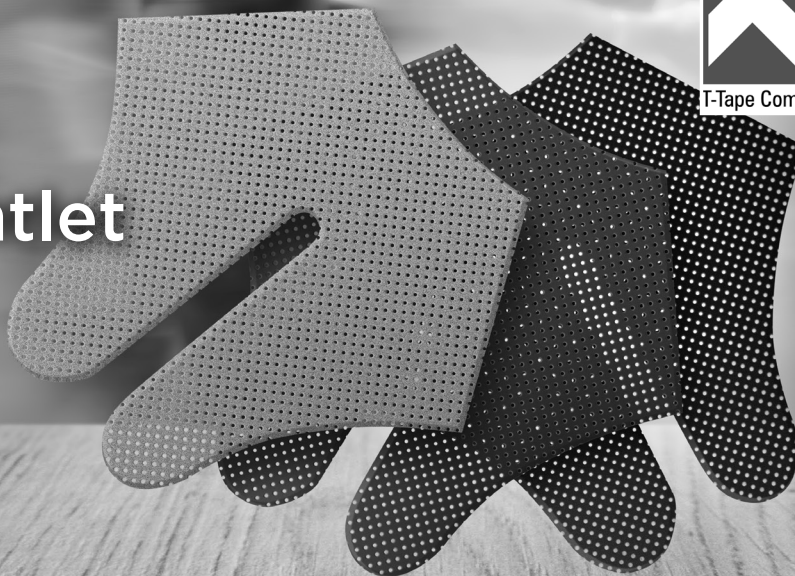

Mediroyal Nordic AB · Box 7052 · 192 07 Sollentuna · Sweden · Tel +46 8 506 766 00
info@mediroyal.se · facebook.com/Mediroyalnordic · www.mediroyal.se

GB

English – instructions

Instruction Thumb Gauntlet

- 1 Immerse the pre-cut in the water and let it stay for about one minute. Carefully lift it up and place it quickly on a dry towel. Move it from the towel onto the patient. Position the cut-out of the splint between the thumb and Dig II. Make sure that there is a slight web-space between the thumb and Dig II. Carefully attach the material together on the outside of the thumb, creating weld. Work the weld with the thumbs until it is tightly fitting the thumb. Then attach the two legs together on the ulnar side of the hand in order to avoid them to be misaligned. While the thermoplastic is still soft, position the thumb into an opposition grip so the tip of the thumb can reach the tip of the Dig II.
- 2 When the thermoplastic has cooled, open the legs on the ulnar side and slide the thumb splint off the thumb. Cut the seam 5 mm from the bottom of the weld. Immerse the weld into the water again for approx 30 seconds until soft but without delaminating. Fold the weld to one side and work the weld into the material until it is flat and comfortable on the inside.
- 3 Adjust the length of the thumb piece by immersing the tip into the water and then fold it circular. Then adjust the length of the two arms by cutting them with a scissor. Round the ends in order to make them more comfortable.
- 4 Adjust the web-space area by immersing it into water and then roll it outwards.
- 5 Attach the hook closure on the part inside of the hand. Cut a hook piece. If not self adhesive, spot heat the surface of the arm for a few seconds then position the hook part and press slightly. The hook part can be welded in the edge by using a strip of thermoplastic. Cut a 3–4 mm wide strip that is 6–8 cm long. Heat it with a heat gun and then attach the adhesive strip around the hook and then work the thermoplastic into the hook part and the main body.
- 6 The loop strap can be permanently welded onto the dorsal flap. Cut a loop strap in the desired length. Use a heat gun to melt the loop on the first 2-3 cm of the strap. Once it has cooled, cut a small hole in the center of the strap. Spot heat the area where the strap should be positioned onto the flat and press it into place. Then let the flap cool. In order to make the attachment stable, cut a thermoplastic piece that is 2–3 mm wider than the strap and 4–5 cm long. Spot heat the first 2 cm of the strip and when soft you position that over the attached loop piece. Cut away the excess part of the strip and work the hot thermoplastic over the strap, pressing the area over the hole in the strap. That creates a thermoplastic weld.

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